Instructions for use

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IMed Consultancy Ltd.
Bloxham Mill, Bloxham OX154FF
United Kingdom

Sword Bloom

Sword Bloom is a medical device software (MDSW) intended to guide individuals through an exercise program directed at pelvic floor muscles and to enable pelvic health specialists to design, monitor and adjust the exercise program based on the individual's needs and progress.

INSTRUCTIONS FOR USE

Doc Ref: SB-IFUP-EN-05

Table of contents

Information for Patients	3
Description	3
Intended Use	4
Intended Users	4
Indications for Use	4
Contraindications	4
Warnings and Precautions	5
Instructions for Use Symbols	7
Help and Support	8
What does the app do?	8
How do I use the app?	9
Perform pelvic floor exercises	9
Login	9
Homescreen	9
Pelvic Sessions	9
Sword Bloom App module Updates	12
Chat with your Women's Health Specialist	12
Learn more about your health condition	12
Customize the Sword Health app	12
Troubleshooting	12
What do I do if the app is not working?	12
What do I do if I can't connect the Pod?	13
Can the Sword Bloom provide a patient diagnosis?	13
Can I send a message to my Women's Health Specialist?	13
Can I send a message to the Sword Health Support team?	13
How do I ask for a recalibration?	13
Can I cancel a session?	13
How do I see the Bloom Pod serial number?	13
How long do I have to use the app?	14
What happens when my session ends?	14
Who can see my data?	14
How do you protect my data?	14
Who can I contact with questions about my data?	15
Liability	15
Warranty	15

Information for Patients

Description

Sword Bloom guides individuals through customised exercise programs to restore or improve movement, function and muscle strength of the pelvic floor. The medical device software provides program instructions and feedback on patients' progress through the Sword Bloom App module supported by Artificial Intelligence (AI) technologies, and by the patient's Pelvic Health Specialist. The exercise program that patients follow is designed, monitored, and adjusted by Pelvic Health Specialists on the Clinical Portal, with the support of AI technologies.

Sword Bloom is comprised of the following:

- Sword Bloom App module - A mobile application used by the member to receive instructions, record exercises being performed, as well as receive and provide feedback through the mobile application itself, Phoenix AI agent and their pelvic health specialist remotely;
- Clinical Portal - A web-based portal that the member's Pelvic Health Specialist uses to assign exercises, monitor the member's performance, review and act on AI-generated insights and suggestions and provide feedback. The Clinical Portal includes the Phoenix AI agent for AI-assisted patient management.

Both user interfaces use Phoenix AI system, an integrated AI/ML system. Phoenix is not a standalone product; all AI capabilities operate within the Sword Bloom App module and Clinical Portal.

Sword Bloom has the following accessory and component, which are supplied together in the Bloom System kit:

- Bloom Pod: an intravaginal pelvic device used to track pelvic floor muscle movement and tone. The values collected by the device are sent to the app to allow for real-time feedback on performance;
- Lubricant: an optional water-based component that is supplied for lubrication.

The Bloom System also contains a pod case, which is a washable fabric bag used for the storing of the Bloom Pod after cleaning.

Sword Bloom can be used without the intravaginal pelvic device through no-pod sessions. Bloom No-Pod sessions offer a way for members to access the pelvic program using only a mobile device without the pod. These sessions are integrated in the Sword Bloom App module and contain exercises that align closely with the pod program. The option to complete no-pod sessions allows members to continue progressing through the Bloom program and maintain consistency even when without the use of the Bloom Pod.

Note: The terminology patient/member/user is used interchangeably within this document. Likewise, the terms Pelvic Health Specialist (PHS) and Women's Health Specialist (WHS) are used interchangeably.

Intended Use

Sword Bloom is a medical device software (MDSW) intended to guide individuals through an exercise program directed at pelvic floor muscles and to enable pelvic health specialists to design, monitor and adjust the exercise program based on the individual's needs and progress.

Intended Users

Sword Bloom is intended to be used by individuals with vaginal anatomy above the age of 18.

It can be used both by individuals who have been recommended a course of exercise by a suitably qualified professional with experience in pelvic physical therapy but also by individuals who have not been affected by specific injury or disability related to pelvic floor issues.

Sword Bloom is intended to be used by lay persons in a non-clinical (home) environment with sufficient space to perform the exercises. No specific education or training is required prior to using the device, as the app provides all necessary use-related instructions.

Indications for Use

Sword Bloom is primarily used to restore movement, function and muscle strength of the pelvic floor. Sword Bloom can also be used outside of its medical purpose by healthy individuals who have not been affected by injury, illness or disability for general health and wellbeing purposes, to work on improving the function of their pelvic floor.

Contraindications

Sword Bloom is not designed to be used by members presenting with any of the following conditions:

- Symptoms/signs compatible with active pelvic infection.
- Symptoms/signs compatible with acute and serious underlying medical conditions (neurological, cardiac, respiratory, gastrointestinal, genitourinary, metabolic or other) precluding participation in an exercise program.

- Respiratory, cardiovascular or metabolic pathology where light to moderate physical exercise exertion is medically contra-indicated.
- Bone fractures or other musculoskeletal conditions in which mobilization of pelvic floor muscles or lower limbs is contraindicated.
- Aphasia, dementia or any other psychiatric comorbidity that interferes with the use of the medical device.
- Complicated or high-risk pregnancy
- Pelvic health complaints related to conditions outside of the program's coverage: i.e.- untreated, active pelvic inflammatory disease (PID).

In addition, the use of Sword Bloom is contraindicated in the presence of continuity solutions or other local complications affecting the vagina.

Warnings and Precautions

Only use Sword Bloom for its intended use as described in this manual. It should not be used as a treatment or preventive measure for any medical conditions. If you have any doubts regarding its use, it is advised to consult with a licensed healthcare professional.

Before each use, examine both the Bloom Pod and the lubricant packaging (tube or sachet) for any visible damage, such as tears, punctures, or leaks. Ensure any tamper-evident seals or shrink-wrap are intact and have not been removed or altered. If you notice unusual changes in the lubricant, such as cloudy color, separation, or an off odor, do not use it. Should you find any damage or evidence of tampering, discontinue use and contact our Support Team.

This is a single-user product. The use of this device by more than one person may present a health risk.

Do not use the Sword Bloom if:

- You have given birth within the last six weeks.
- You have a complicated or high-risk pregnancy.
- You have a medical condition affecting your pelvic health (for example, abnormal vaginal discharge or severe prolapse) and have not spoken to your Bloom pelvic health specialist or have not been cleared by a medical provider.

Do not use the Bloom Pod if:

- You are pregnant.
- You experience persistent pain when inserting or using the Bloom Pod.
- You are allergic to medical-grade silicone.

- The Bloom Pod has been modified, damaged, and/or tampered with.

The Bloom program is not designed to be used by those with a complicated or high-risk pregnancy. If you wish to use the Bloom program during your pregnancy, you should first consult with your physician or other qualified health provider prior to participation. Insertion of the Bloom Pod during any stage of pregnancy is not a requirement for participation in the program and is discouraged.

Do not leave the Bloom Pod inside your vagina for more than half an hour at a time.

The Bloom Pod is made with medical-grade silicone. However, if you experience skin and/or vaginal irritation, symptoms/signs compatible with urinary tract infection (UTI), or any other unexpected symptom, discontinue use immediately and consult a healthcare professional. For safe operation and to help prevent skin and/or vaginal irritation or UTIs, please read and carefully follow the cleaning instructions.

During normal use, the temperature on the applied part of the Bloom Pod can reach up to 41.5°C.

The Bloom Pod is completely covered in medical grade silicone with no exposed parts, being safe to use with most intrauterine devices (IUD). If you have an active implanted device that would be affected by a magnetic field (e.g. pacemaker, IUD, implantable cardioverter defibrillator), it is recommended that you consult with a licensed healthcare professional before using the Bloom Pod.

The Bloom Pod is safe to use during menstruation. As with general usage, follow the cleaning guidelines. Additionally, throughout your menstrual cycle, your intravaginal pressure range may vary - it's just a natural fluctuation in your strength as part of your monthly cycle.

Bloom Pod is not a substance-based device and the materials used are compliant with REACH, Prop65, and RoHS.

Use of the Bloom Pod adjacent to other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

The Bloom Pod uses Radio Frequency (RF) energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The Bloom Pod is suitable for use in all establishments, including domestic establishments.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Bloom Pod. Otherwise, degradation of the performance of this equipment could result.

Bloom Pod was tested against the emission requirements of IEC 60601-1-2, being classified as Class B, Group 1. It passed the testing for all of the following: Radiated RF Emissions; equipment Immunity testing for EN 61000-4-2 Electrostatic Discharges and

the IEC 61000-4-3 Radiated RF EM Fields and IEC 61000-4-3 Proximity Wireless fields, IEC 61000-4-8 Power frequency magnetic field, IEC 61000-4-39 Radiated fields in close proximity immunity test with levels defined for home healthcare environment in accordance with IEC 60601-1-2 2015 and IEC 60601-1-2 AMD1_2020.

Instructions for Use Symbols

Explanation of the symbols on the Sword Bloom IFU and on the label	
	Consult Sword Bloom Instructions for Use
	Manufacturer
	Date of Manufacture
	Catalogue Number
	Batch Code
	Caution, there are specific warnings or precautions associated with the device which are not found on the label – consult the instructions for use
	Medical Device

	Unique Device Identifier
	UKCA Marking under UK MDR 2002
	UK Responsible Person (UKRP)

Help and Support

In the event of any serious incident occurring in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.

For patients, please communicate this through your Women’s Health Specialist who will notify Sword Health (the manufacturer) directly without delay.

For assistance, you can also email our Support team at help@swordhealth.com (in the US), helpuk@swordhealth.com (in the UK) or helpca@swordhealth.com (in Canada).

If you require a printed copy of these Instructions for Use, please contact Sword Health’s Support team. It will be provided to you within 7 working days without any additional cost.

The product label can be found under the “About” menu section.

What does the app do?

The Sword Health App, which includes the Sword Bloom App module, is a native mobile application that connects to the Bloom Pod, processes the data it collects, and provides real-time feedback to guide you through your exercises. The App will provide cues with corrective guidance to help you perform the exercises properly.

You can also provide voice feedback on the exercises performed and on the session in general to Phoenix, our AI agent, during or after the session. This feedback will then be provided to your appointed WHS.

The Sword Bloom App module also syncs your session results to the Clinical Portal. This allows your Women's Health Specialist to monitor your progress and adjust your program as needed. In order to fully operate, the Sword Bloom App module requires an active internet connection (via Wi-Fi) to synchronize your data. The smartphone running the app (Android version 8 or newer / iOS16 or newer) must have Bluetooth Low Energy (BLE) enabled to connect and communicate with the Bloom Pod.

How do I use the app?

Perform pelvic floor exercises

Download the Sword Health app to connect your Bloom Pod. Open the app.

Login

Use your account credentials to login (email and password defined during the enrolment flow when you decide to start your program).

1. If you forget your password: choose the option "Forgot password?", the app will guide you through a flow where you will receive an email to update your password. After password re-definition you can login.
2. If you insert an incorrect password several times your account will be temporarily locked.

Homescreen

The home screen of the Sword Bloom App module shows the member care journey path, results of past sessions, personal goals, and an entry point to perform exercise sessions.

Depending on the program in which you are enrolled, the home screen may also display additional relevant exercises, such as movement and mindset, assigned by your WHS.

Pelvic Sessions

You may complete an interactive pelvic baseline session, including questions and no pod exercises, for your Women's Health Specialist to consider when creating your program plan.

After the plan is designed by your Women's Health Specialist, it will be displayed on the Sword Bloom app module homepage. When starting a pelvic session, you can choose between starting a session with the Pod or starting a session without the Pod.

Exercise position:

- Before starting the pelvic session, the Sword Bloom App module will guide you on the appropriate exercise position.
- To adjust the level of difficulty, exercises may be performed in different positions as instructed in the App.

In case you perform a Pod session:

- The Bloom Pod should be inserted into the vagina.
- Ensure the tail of the Bloom Pod remains outside of the body at all times.
- Before use, please consult the printed Instructions for Use provided with the Bloom Pod and follow all general guidelines provided by Sword Health.

Pelvic floor movements:

- For pelvic contraction, imagine that you are trying to stop yourself from peeing. In contraction, the pelvic muscles are lifting inwards and upwards.
- Pelvic relaxation is the period of rest between lengthening and contracting. Think of it as “letting go”.
- For pelvic lengthening, imagine that you are trying to start a flow of urine or pass gas. In lengthening, the pelvic muscles are dropping down and stretching away.
- When exercising, remember to:
 - Maintain a relaxed stomach, buttocks and thighs.
 - Try not to hold your breath.
 - Breathe in when lengthening your pelvic floor.
 - Breathe out when contracting your pelvic floor.

The Sword Bloom App module provides real-time guidance during sessions, including instructions on how to use the Bloom Pod, if applicable, and how to correctly perform each exercise.

(For Pod sessions only) The The Sword Bloom App module provides detailed guidance on the use of the Bloom Pod, including:

- (i) cleaning instructions to be followed before and after each use;
- (ii) connection instructions;
- (iii) insertion and removal instructions; and
- (iv) calibration instructions to allow the app to understand the relaxation baseline for the member, as well as their maximum contraction range.

The Sword Bloom App module has a session preview screen that shows you a summary of the activities and exercises of the session.

During the exercises, the app will display instructions/games, along with animations, to guide you through the exercise.

For Pod sessions, performance is tracked during the exercises through the data collected by the intravaginal Pod. Real-time feedback and additional instructions are given by the app to you to support correct exercise execution. You can also provide feedback on the exercises in particular and the session in general to Phoenix, by voice, which will then be transmitted to the WHS. Make sure you are in a quiet environment, so Phoenix can better interpret your voice.

While performing the exercises, some actions are available when clicking on the pause button, such as:

- i. Pelvic floor instructions: Shows the user a set of screens explaining how to contract, relax and lengthen the pelvic floor.
- ii. Audio instructions: Allows the user to turn on and off the audio instructions.
- iii. (Pod sessions only) Exercise instructions: Shows the exercise preview screen.
- iv. (Pod sessions only) Exercise tutorials: Shows the exercise tutorials, if available for that exercise.
- v. (Pod sessions only) Sound: Allows the user to turn on and off the audio from the game.
- vi. (Pod sessions only) Skip exercise: Allows the user to skip the exercise. In this case a reason must be given.
- vii. (Pod sessions only) Restart exercise: Allows the user to restart the exercise.
- viii. (Pod sessions only) Recalibrate: Allows the user to perform the calibration step again.
- ix. (No Pod sessions only) Vibration: Allows the user to turn on and off the vibration.

(For Pod sessions only) When the last exercise of the session is completed, the session results are shown. A session details page will show the overall performance of the session and detailed performance on each individual exercise.

At the end of the session a quality questionnaire is displayed for you to rate the session experience: questions on symptoms, fatigue and overall session quality.

Sword Bloom App module Updates

If you use your app on a regular basis, it is likely that you will receive updates when the session starts. This process doesn't require user interaction.

These updates can include improvements to the Bloom Pod, therefore you shall keep the Pod close to the phone until the update is complete.

Chat with your Women's Health Specialist

Your Women's Health Specialist is available through the chat in the App.

Learn more about your health condition

Academy is a module of the Sword Health App with educational videos on health conditions, exercise, nutrition, sleep and overall wellbeing.

Customize the Sword Health app

On the profile page of the Sword Health app the user can:

- Set reminders to perform sessions
- Configure notifications
- Configure language

The user can also access the legal documents and FAQs, contact support, create a pin code, share a testimonial, log out amongst others.

Troubleshooting

What do I do if the app is not working?

If you have trouble navigating through the Sword Health app, setting up your Pod or using it, email our Support team at help@swordhealth.com (in the US), helpuk@swordhealth.com (in the UK) or helpca@swordhealth.com (in Canada). You can also message your Women's Health Specialist on the "Chat" tab on the Sword Bloom App module within the Sword Health app.

What do I do if I can't connect the Pod?

If you're having trouble connecting your Pod, please contact us at help@swordhealth.com (in the US), helpuk@swordhealth.com (in the UK) or helpca@swordhealth.com (in Canada).

Can the Sword Bloom provide a patient diagnosis?

No. The session results analysis is performed by a Women's Health Specialist, who concludes the progress/effectiveness of a certain pelvic health routine adjusted to the patient's diagnosis.

Can I send a message to my Women's Health Specialist?

You can message your Women's Health Specialist on the "Chat" tab on the Sword Bloom App module within the Sword Health app.

Can I send a message to the Sword Health Support team?

You can message our support team through the profile page of the Sword Health app or by sending an email to help@swordhealth.com (in the US), helpuk@swordhealth.com (in the UK) or helpca@swordhealth.com (in Canada).

How do I ask for a recalibration?

You can request a recalibration at any time during an exercise. This option is available in the pause menu.

If you are not able to perform the calibration by yourself, Phoenix will support the process.

Can I cancel a session?

You have an option to leave the session before it starts, clicking on the back arrow buttons at the top of the screen.

How do I see the Bloom Pod serial number?

To see the Bloom Pod's serial number the member needs to navigate to the Profile tab, scroll down to the Help section and tap the Pod serial number option, from where a new screen is opened that shows the serial number of the pod.

You can also find the Bloom Pod serial number in the Bloom Pod package inside your kit.

How long do I have to use the app?

The goal of the app is to guide you through an exercise program that can improve your quality of life and tackle pain, discomfort or disability related to your pelvic floor. Because everyone has different goals and needs, the length of your program can vary. Your Women's Health Specialist will help you determine how long you need to use the app for. Users of the app typically start noticing improvements as early as 3-5 sessions, but 9 sessions are usually the minimum number of sessions that we recommend.

What happens when my session ends?

The app sends your session results and feedback provided to your Women's Health Specialist. The results will be analysed and your sessions will be adapted according to your performance.

Bloom Pod Troubleshooting

Please refer to the printed Instructions for Use included in your kit for troubleshooting information specific to the Bloom Pod.

Who can see my data?

Your data is shared with the Women's Health Specialist team who provides your care. Only authorised people are allowed to see your data.

The data that they hold includes:

- Data about you and your clinical information
- Data about your session (pelvic movement detection and feedback provided by you)

How do you protect my data?

We protect your data in several ways:

- Data is stored at rest and in backups using AES 256 encryption.
- Sword utilises the cloud provider's server-side encryption. The cloud provider manages keys.

- Access to data will be granted only to authorised users who are legally cleared to see data and on a 'need to know' basis.
- Security monitoring mechanisms are in place.

Users also play a critical role in protecting your data. They must follow secure password practices and report suspected security incidents immediately.

Who can I contact with questions about my data?

You should contact your Women's Health Specialist initially. If you have any questions for Sword Health please contact our dedicated data protection officer by sending an email to dpo@swordhealth.com.

Liability

In no event will Sword Health S.A., or any of its affiliates, be liable regarding misuse of Sword Bloom, including negligence or uses not described in this document. Whilst being a device for restoring or improving movement, function and muscle strength of the pelvic floor, use of Sword Bloom does not automatically guarantee patient improvement. Every member using Sword Bloom must be under the care of a Women's Health Specialist.

Warranty

After the delivery to the end user, the responsibility of Sword Health S.A. will be limited to supporting the use of the Sword Bloom and the replacement of faulty accessories or components. Sword Health S.A. reserves the right to refuse warranty coverage, and will not be responsible for replacing any accessory or component nonconforming or defective under this warranty if:

- any of the supplied parts is altered, tampered, not maintained and/ or not used as specified in this guide or
- presents non-compliance or defect that resulted from any technical service or equipment maintenance executed by personnel not authorized by Sword Health S.A.

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©Sword Health S.A. All Rights Reserved. Tous droits réservés.